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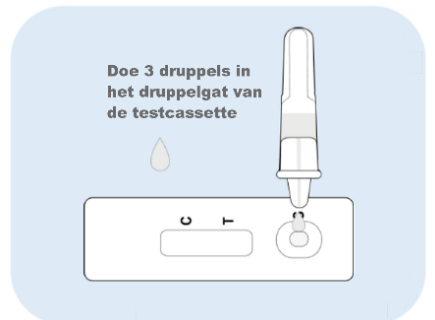
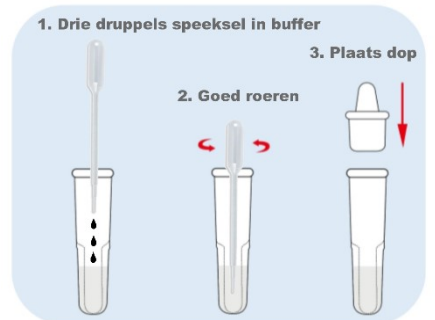
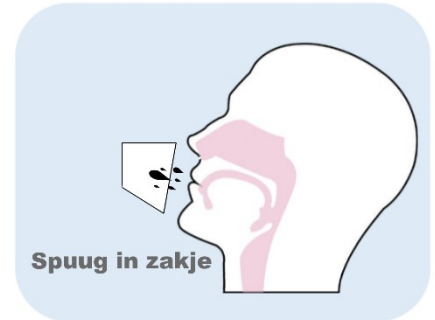
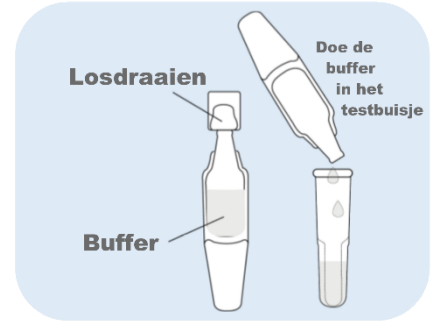
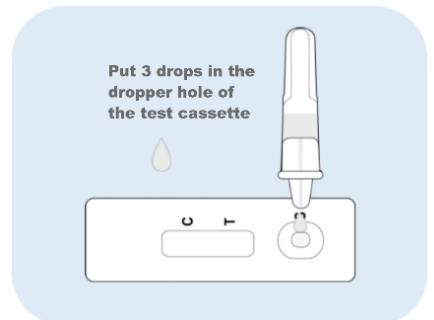
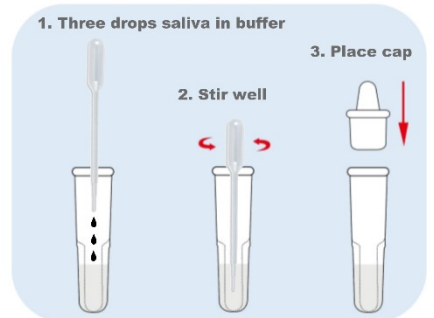
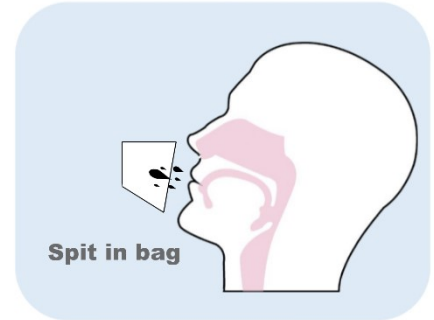
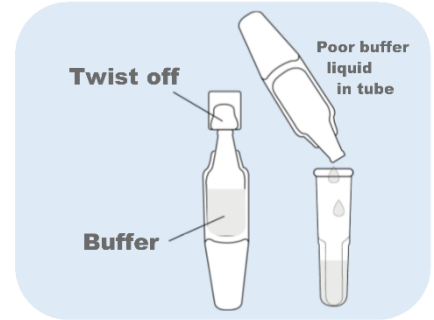
Content: Joysbio Test cassette, test tube with cap, buffer liquid, saliva sample bag, pipette, instructions for use.
This guide is not a replacement for the technical instructions for use (IFU).

1. Blow your nose and wash your hands with soap. Dry your hands.
2. Place the test tube vertically in a tube holder and put the buffer liquid inside the test tube.
3. Remove the rapid test cassette from the foil pouch. Place it on a horizontal base.
4. Prepare the saliva sample bag.
5. Place the saliva sample bag over the mouth. Collect the saliva sample by spitting into the bag. If there is not saliva directly in the mouth, the cheeks can be massaged for 15-30 seconds.
6. Place the tip of the pipette in the saliva sample bag and draw up some saliva. Place three (3) drops of saliva in the test tube, keeping the dropper tip in the tube until just above the buffer solution. Squeeze the remainder of the saliva sample from the pipette back into the saliva bag.
7. Using the pipette, **thoroughly** stir the buffer solution and the saliva sample for 20 seconds. Place the cap on the test tube.
8. Put three (3) drops of the buffer liquid with the sample into the dropper hole of the cassette (near S).

9. Wait 15 minutes to read the test result. Read the test result between 15 and 20 minutes. Any test result after 20 minutes is invalid.

When the test result is positive

Go into home isolation and avoid contact with other people. Contact the public health authority and have yourself tested at a testing facility with a PCR test. If you need more care, please contact your doctor or GP.

When the test result is negative

Stick to the rules for keeping your distance, hygiene and wearing face masks where they are required. A negative test result is not a complete guarantee that the virus is not detected. In some cases the rapid test cannot detect low viral concentrations.

Test result