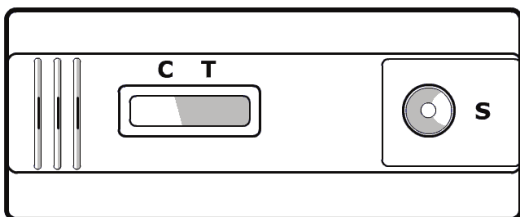
Sensitest®

©Sensitest - LH Cassette

- Ovulatietest met hoge gevoeligheid
- Ovulation test with a high sensitivity.

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Directions for use Sensitest Ovulation test cassette (LH)

Carefully read the directions for use before you start testing. The Sensitest ovulation test is a simple way of ascertaining a woman's 2 to 3 fertile days in her cycle. Some 24-48 hours before ovulation commences, the quantity of Luteinizing Hormone (LH) increases in the urine. Immediately before ovulation, the measurable quantity of this hormone in the urine rises sharply. This LH peak lasts for 1 or 2 days only. The Sensitest ovulation test will indicate 2 clear lines (image 3). After the LH content in the urine is at its highest, ovulation takes place within 24 to 48 hours. You have the best chance to become pregnant during this period, following sexual intercourse. To find this LH peak, you will have to test for several consecutive days. You will first have to determine your cycle length. Take an average of the past cycles. The best day for starting to test can be read from the table (image 4). You count from the first day of your menstruation which is day 1. Continue testing until you can read the LH peak from your test. Ovulation then occurs within 24 to 48 hours. After the LH peak you can stop using the ovulation test.

Timing

The test can be carried out at any time of day. Preferably not with morning urine. However, it is important that testing takes place at about the same time on all consecutive days.

What you need before you start testing

1. A sealed foil package with a Sensitest ovulation test cassette. Do not use an ovulation test from an opened pack.
2. A clean, dry receptacle or plastic cup.
3. Urine from the woman whose ovulation must be confirmed.

The test in 7 steps

1. Take a closed pack and check that it indicates "Ovulation test". Do not use a test that has passed its shelf life.
2. Collect a little urine in a clean receptacle or plastic cup.
3. Open the pack with the test with dry, clean hands.
4. Place the test on a flat, dry base.
5. Using the sample dispenser (pipette), withdraw some urine from the cup.
6. Slowly dispense 6 drops of urine into the hole of the cassette marked with an S.
7. Wait 5-10 minutes before reading the result. It is important that the background is no longer discoloured. However, do not wait more than 30 minutes before reading the result.

Test result

- The result is **negative** if a single line appears (image 1). This is the control line (C) to see whether you have carried out the test correctly. If the test line (T) is not visible or appears as a light pink line (image 1 and 2), then the body has made insufficient LH for an LH peak to be reached. Both test results can be interpreted as negative.
- The result is **positive** if two lines appear (image 3). You can clearly see a control line (C) and a test line (T). Both lines are of the same color or the test line (T) is darker than the control line (C). The level of LH in your body is at its highest at this point.

Result no lines

- If no lines are visible, the test has probably been carried out incorrectly. The cassette has possibly absorbed too little urine. Again dispense a few drops of urine into the hole (S).

If the result is negative

If the ovulation test cannot show an LH peak after repeatedly testing on consecutive days, it is possible that you have carried out the test on the wrong days. Calculate the average length of your cycle again. On your next cycle begin testing two days earlier than the table (image 4) indicates. If you obtain negative results after repeated attempts, we advise to contact your family doctor or gynaecologist. Consult a doctor if irregular or unusually long cycles are experienced.

If the result is positive

If the test indicates that the LH level in your urine is at its highest, ovulation will take place within 24 to 48 hours. You have the best chance of becoming pregnant during this period, following sexual intercourse.

Restriction 1: Reliability of the ovulation test

With a 99% reliability, the Sensitest ovulation test cassette is just as reliable as a comparable test by your family doctor or gynaecologist. However, your personal medical situation may influence the result of this test. Women with polycystic ovary syndrome may obtain an incorrect positive result. In case of doubt, consult your family doctor or gynaecologist.

Restriction 2: Do not use alcohol

Alcohol may influence the test result. Do not use ovulation tests after taking alcohol.

Restriction 3: Don't use as a contraception method

Don't use the ovulation test as a contraception method. Since sperm can remain active for a number of days, the chance of becoming pregnant remains.

Restriction 4: Applying the test

This test may be used only with collected urine. If you require an ovulation test to hold in the urine stream, use the Sensitest ovulation test midstream.

Restriction 5: Do not drink too much

Drinking large amounts of water or other liquids like coffee or tea could dilute your urine and produce light or unclear results. You are advised to restrain from drinking large quantities of liquids during your testing period. Preferably stop drinking 2 hours before testing.

Restriction 6: Ovulation test and the pregnancy hormone (hCG)



The ovulation test gives a positive test result on the pregnancy hormone hCG. If you have a positive test result on your ovulation test during 4 or 5 consecutive days with a dark purple T-line (like image 3), you could consider doing a pregnancy test first. Up to 9 weeks after a miscarriage you could expect positive test results on an ovulation test as well.

Restriction 7: Sensitivity


Each type or brand ovulation test has its own sensitivity. Do not use different types or brands of ovulation tests simultaneously, because test results may vary.

The text of these directions for use was revised on 29 October 2013 Version 1.5

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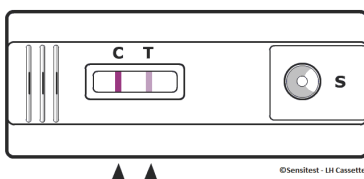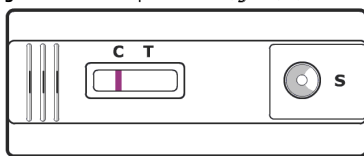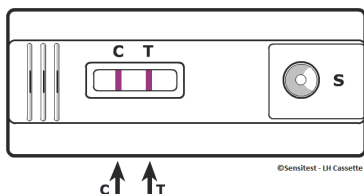For in vitro diagnostic use. For external use only, do not swallow. Keep out of reach of children.

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