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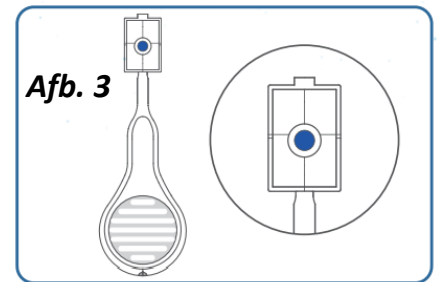
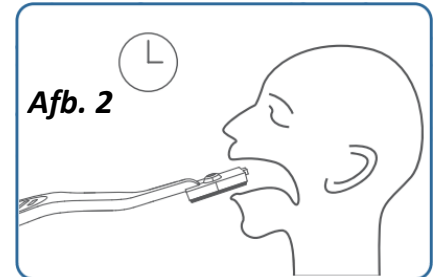
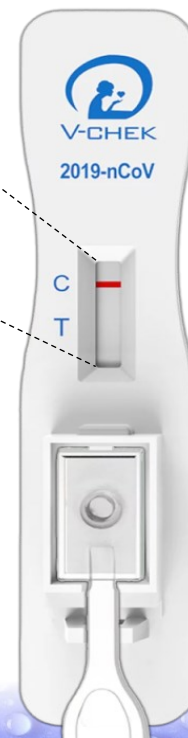
Jouw
test

V-CHEK QUICK GUIDE - TIPS AND TRICKS COVID-19 ANTIGEN LOLLIPOP SELFTEST (EN)

Content: V-Chek Test cassette, Lollipop saliva stick, Instructions for use.
This guide is not a replacement for the technical instructions for use (IFU).

1. Wash your hands with soap. Dry your hands. Do not eat, drink or smoke at least 30 minutes before testing.
2. Make sure the test parts are at room temperature. Remove the V-Chek self-test cassette from the foil pouch. Place the test cassette on a horizontal surface (*image 1*).
3. Remove the lollipop stick from the foil pouch.
4. Take the saliva sample by holding the stick side with the sponge in the mouth. Actively wipe the inside of the mouth and tongue to collect saliva. Do not bite the sponge (*image 2*).
5. Continue this for approximately 90 seconds until the sponge is soft or the indicator tip on the sponge turns blue (*image 3*).
6. Press the sponge side of the saliva stick into the rectangular hole of the test cassette. Click the stick into the holder (*image 4*).
7. The saliva containing the antigen sample is now absorbed from the sponge by the test.

Image 1

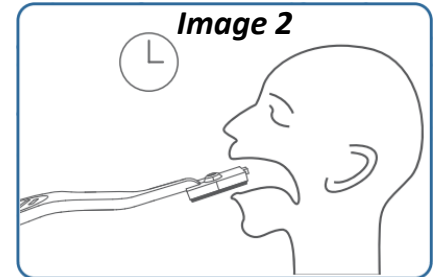
Image 2

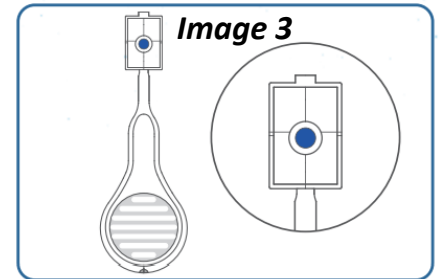
Image 3

8. Wait 15 minutes to read the test result. Read the test result between 15 and 20 minutes. Any test result after 20 minutes is invalid.

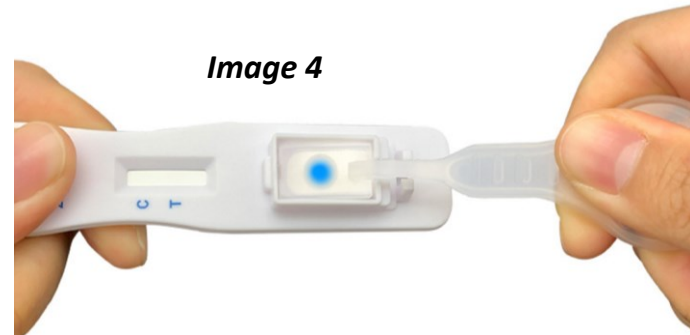
Image 4

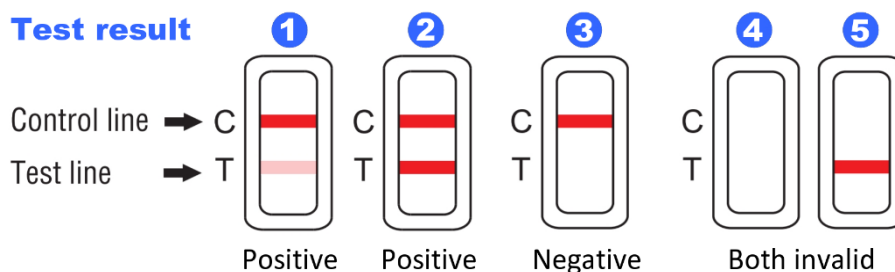
When the test result is positive

Go into home isolation and avoid contact with other people. Contact the public health authority and have yourself tested at a testing facility with a PCR test. If you need more care, please contact your doctor or GP.

When the test result is negative

Stick to the rules for keeping your distance, hygiene and wearing face masks where they are required. A negative test result is not a complete guarantee that the virus is not detected. In some cases the rapid test cannot detect low viral concentrations.

Test result

Your test